



Open Foot
Premium Heel Protector



Closed Foot
Heel Protector



HCS HEEL PROTECTOR BOOT WITH ATTACHABLE WEDGE

The HCS high-performing Heel Protector Boot is designed to provide maximum comfort and protection for individuals at risk of developing heel pressure ulcers or experiencing heel pain. It helps redistribute pressure, reduce friction, and maintain proper alignment to promote healing and prevent further injury.

FEATURES

- Pressure relief - designed with a 2" heel lift to elevate the heels above the bed surface and help protect against heel pressure ulcers;
- Application instructions printed on boot;
- Soft and comfortable design – foam cushioning protects lateral and medial malleoli (bony prominences on the ankle);
- The anti-friction antimicrobial outer layer is made of breathable nylon, allowing moisture and heat to migrate away from the foot. The inner-liner material is designed to manage moisture within the boot, keeping the patient's skin and boot liner dry;
- Off-loading heel opening - allows for ventilation and easy monitoring of the heel;
- Foot Drop prevention strap;
- Structured foot rest;
- The Premium Open Foot Gate boot offers improved airflow, temperature control, and easy viewing of the foot;
- Adjustable contracture prevention straps;
- Adjustable calf straps;
- Standard size – 10-18in calf circumference;
- SCD ports openings;
- Optional attachable anti-rotation wedge included to prevent lateral rotation of the foot;
- Latex-Free.

PRECAUTIONS

- Intended for use in bed; DO NOT stand or walk while wearing heel protector;
- Remove the heel protector as required to assess skincare;
- Check SCD tubing ports after turning and/or transferring;
- Use dedicated ports for tubing; do not use heel opening ;
- If redness appears around tubing contact your wound care expert;
- Read instructions for use prior to using the heel protector.

INTENDED USE

- Pressure injury prevention and care of the heel;
- Reduce plantar flexion risk;
- Helps prevent lateral rotation of the leg and foot;

HCS HEEL PROTECTOR INSTRUCTIONS



APPLY

Open the heel protector and position the patient's heel into the opening. Ensure the patient's heel is centered and offloaded. You should be able to see the patient's heel through the opening.



SECURE

Secure straps around the heel protector, ensuring they are tight across the patient's foot to prevent heel drop. **DO NOT** over tighten the straps. *Attach straps following steps 1-4 as indicated on boot.*



PLACE

Place the anti-rotation wedge tight against the side of the heel protector, positioning it either on the medial or lateral side to prevent internal or external rotation of the foot and knee.



CONFIRM

Position your hand under the heel protector to verify that the heel is offloaded from surface and that the straps are secure.

To Use with Sequential Compression Device: run the tubing through the side ports on either side of heel protector. Ensure tubing is connected outside the heel protector and the tubing does not run up through the heel protector.



OPEN

HCS Premium Heel Boot- Open Foot Gate
Use the open foot gate for quick and easy skin assessments and to allow airflow without removing the heel protector.



REMOVE

For removal, gently release the straps from the heel protector and remove the wedge.


CLEANING INSTRUCTIONS

The HCS Heel Protector is single-patient use, but can be maintained by cleaning with an approved by your facility disinfectant cleaner. The device is machine washable and dried in a dryer on low temperature.

RELATED PRODUCTS



#82200 Protector Heel Convoluted Foam

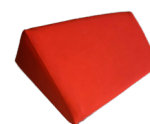

#82203 Wedge for Heel Protector Boot w/Velcro

ORDERING INFORMATION

MFG#	CHS#	Description	UMO
82200	408267	Protector Heel Convoluted Foam	36 pairs/case
82201	408268	Protector Heel Boot Closed Foot w/Wedge, Standard size (10-18in)	12 ea/cs
82202	408269	Protector Heel Boot PREMIUM, Open Foot w/Strap & Wedge, Standard size (10-18in)	12 ea/cs
82203	409665	Wedge for Heel Protector Boot w/Velcro Accessory	1 ea



Concordance Healthcare
Solutions, LLC.

T (800) 447-0225
W concordancehealthcare.com

85 Shaffer Park Drive Tiffin, OH 44883